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Product

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Instructions for use

The Product

Derma*gi*q® skin
30 g cream for the treatment of wounds caused by eczema and other skin disorders

Ingredients

Arachis oleum, cera alba, glyceryl oleate, mel, aqua

Product description

Derma*gi*q® skin is an easy to apply ointment for the treatment of superficial open wounds caused by eczema and other skin disorders (including the redness and itching caused by insect bites). Derma*gi*q® skin combines a skin-friendly base cream combined with pure honey, and can be used to treat large areas of skin. The honey used in Derma*gi*q® skin is rich in honey enzymes, notably glucose oxidase. In a moist environment, glucose oxidase produces small amounts of hydrogen peroxide. Thanks to the high sugar concentration in the honey, some of the moisture from the wound mixes with Derma*gi*q® skin to form a protective antibacterial layer. Derma*gi*q® skin contains no traces of pesticides, preservatives or other substances that are known to cause allergic reactions.

Instructions for use

1. Use the cap to pierce the seal of the tube
2. Apply a thin layer of Derma*gi*q® skin to the areas to be treated
3. Gently massage Derma*gi*q® skin
4. Allow 10 minutes for Derma*gi*q® skin to be absorbed before covering, if desired, with a gauze or dressing, or with clean, non-irritant clothing
5. Treat the affected area once to three times a day

Indications

Derma*gi*q® skin can be used on superficial and open wounds such as those caused by eczema. Derma*gi*q® skin can also be used to soothe skin that is irritated, red, or itchy, or to protect intact, dry, or damaged skin. Derma*gi*q® skin is suitable for all ages.

Contra indications

Do not use Derma*gi*q® skin if the patient is known to be allergic to honey.

Side effects

- During the first minutes after application on minor wounds Derma*gi*q® skin can cause a light stinging sensation
- In case of unexpected (not-mentioned) side effects after usage of Derma*gi*q® skin, contact the distributor or manufacturer.

Safety precautions

- Do not use Derma*gi*q® skin if the tube is damaged
- Always replace the top on the tube after use
- Do not allow the tube to come into direct contact with the surface of the wound
- Once the tube has been opened use the contents within 3 months
- Do not treat different patients with the same tube

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