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Product

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Instructions for use

The Product

Dermagiq® skin
30g/50g cream for the treatment of wounds caused by eczema and other skin disorders

Ingredients

Achris oleum, cera alba, glyceryl oleate, mel, aqua

Product description

Dermagiq® skin is an easy to apply ointment for the treatment of superficial open wounds caused by eczema and other skin disorders (including the redness and itching caused by insect bites). Dermagiq® skin combines a skin-friendly base cream combined with pure honey, and can be used to treat large areas of skin. The honey used in Dermagiq® skin is rich in honey enzymes, notably glucose oxidase. In a moist environment, glucose oxidase produces small amounts of hydrogen peroxide. Thanks to the high sugar concentration in the honey, some of the moisture from the wound mixes with Dermagiq® skin to form a protective antibacterial layer. Dermagiq® skin contains no traces of pesticides, preservatives or other substances that are known to cause allergic reactions.

Instructions for use

1. Use the cap to pierce the seal of the tube
2. Apply a thin layer of Dermagiq® skin to the areas to be treated
3. Gently massage Dermagiq® skin
4. Allow 10 minutes for Dermagiq® skin to be absorbed before covering, if desired, with a gauze or dressing, or with clean, non-irritant clothing
5. Treat the affected area once to three times a day

Indications

Dermagiq® skin can be used on superficial and open wounds such as those caused by eczema. Dermagiq® skin can also be used to soothe skin that is irritated, red, or itchy, or to protect intact, dry, or damaged skin. Dermagiq® skin is suitable for all ages.

Contra indications

Do not use the product when the patient is known to be sensitive to one of the ingredients of the ointment (see Ingredients). Hypersensitivity to one of the ingredients of the ointment can cause redness, irritation and possibly light pain.

Side effects

- During the first minutes after application, the product can cause a light stinging sensation. The stinging sensation is probably caused by the low pH and high osmolarity of the honey and has no effect on the performance of the product.
- In the event of unexpected side effects after use, contact your doctor or the manufacturer. Side effects can cause inconvenience to the patient.

Safety precautions

- Do not use if the tube is damaged as this can lead to contamination of the product with bacteria and to wound infection. Because of the antibacterial protection that honey provides, the chance of wound infection is considered very low.
- Always replace the top of the tube after use. An open tube could lead to contamination with bacteria and wound infection. Because of the antibacterial protection that honey provides, the chance of wound infection is considered very low.
- Do not allow the tube to come into direct contact with the surface of the wound. Direct contact between the wound/skin and the uncapped tube could lead to contamination with bacteria and wound infection. Because of the antibacterial protection that honey provides, the chance of wound infection is considered very low.
- Do not apply this product for more than 30 days continuously. Treatment for more than 30 days continuously could lead to inconvenience to the patient, such as irritation and/or minor pain.
- The contents of the tube must be used within 3 months of opening. Longer use could lead to lower functionality and contamination of the product. Because of the antibacterial protection that honey provides, the chance of wound infection is considered very low.
- Do not treat different patients with the same tube as this can lead to cross-contamination with bacteria and to wound infection. Because of the antibacterial protection that honey provides, the chance of wound infection is considered very low.
- Do not use the product after the expiration date as this can lead to lower performance of the product and to inconvenience for the patient.

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