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## Instructions for use

**Product**  
Dermagiq® Proct perianal ointment  
Dermagiq® Proct perianal ointment (tube, 15 gram) is intended for superficial wounds and minor irritations near the anus.

**Ingrediënten**  
Arachis oleum, cera alba, glyceryl oleate, mel, aqua

**Product description**  
Dermagiq® Proct perianal ointment is an easy to apply ointment for the treatment of superficial wounds and chapping around the anus, as caused by haemorrhoids for example. Dermagiq® Proct perianal ointment consists of a relief-giving ointment and pure, enzyme-rich honey. This combination of ingredients gives the ointment its soothing and healing properties. Dermagiq® Proct perianal ointment also protects superficial wounds against further infection. Once applied, the ointment reduces itchiness and has a slightly cooling effect. Dermagiq® Proct perianal ointment contains no substances that are known to give rise to allergic reactions.

**Indications**  
Dermagiq® Proct perianal ointment is suitable for all ages and can be used to treat:

- superficial wounds around the anus, whether or not infected
- anal fissures (chapping)
- irritation and itchiness around the anus as a result of haemorrhoids etc.
- irritated skin caused, for instance, by incontinence or obesity.

**How to apply**

- Remove the screw cap from the tube
- Use the cap to puncture the seal on the tube
- Squeeze a little ointment onto a fingertip. Apply the ointment by massaging it gently into the skin around, and if necessary in, the anus. If possible, cover completely
- Apply the ointment two to four times a day.

**Contra indications**  
Do not use the product when the patient is known to be sensitive to one of the ingredients of the ointment (see Ingredients). Hypersensitivity to one of the ingredients of the ointment can cause redness, irritation and possibly light pain.

**Side effects**

- During the first minutes after application, the product can cause a light stinging sensation. The stinging sensation is probably caused by the low pH and high osmolality of the honey and has no effect on the performance of the product.
- In the event of unexpected side effects after use, contact your doctor or the manufacturer. Side effects can cause inconvenience to the patient.

**Safety precautions**

- Do not use if the tube is damaged as this can lead to contamination of the product with bacteria and to wound infection. Because of the antibacterial protection that honey provides, the chance of wound infection is considered very low.
- Always replace the top on the tube after use. An open tube could lead to contamination with bacteria and wound infection. Because of the antibacterial protection that honey provides, the chance of wound infection is considered very low.
- Do not allow the tube to come into direct contact with the surface of the wound. Direct contact between the wound/ skin and the uncapped tube could lead to contamination with bacteria and wound infection. Because of the antibacterial protection that honey provides, the chance of wound infection is considered very low.
- Do not apply this product for more than 30 days continuously. Treatment for more than 30 days continuously could lead to inconvenience to the patient, such as irritation and/or minor pain.
- The contents of the tube must be used within 3 months of opening. Longer use could lead to lower functionality and contamination of the product. Because of the antibacterial protection that honey provides, the chance of wound infection is considered very low.
- Do not treat different patients with the same tube as this can lead to cross-contamination with bacteria and to wound infection. Because of the antibacterial protection that honey provides, the chance of wound infection is considered very low.
- Do not use the product after the expiration date as this can lead to lower performance of the product and to inconvenience for the patient.

**Storage conditions**  
Store between 4° en 30°C

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